**medi GmbH & Co. KG**  
**Medicusstraße 1**  
**D-95448 Bayreuth / Germany**  
**T +49 921 912-0**  
**F +49 921 912-780**  
**ortho@medi.de**  
**www.medi.de**

medi Australia Pty Ltd  
 Unit 4/13-21 Hallmark Street  
 Pendle Hill NSW 2145  
 Australia  
 T +61 2 9890 8696  
 F +61 2 9890 8439  
 sales@medi.au  
 www.medi.australia.com.au

medi Austria GmbH  
 Adamgasse 16/7  
 6020 Innsbruck  
 Austria  
 T +43-512 57 95 15  
 F +43-512 57 95 15 45  
 vertrieb@medi-austria.at  
 www.medi-austria.at

medi Belgium NV  
 Staatsbaan 77/0099  
 3945 Ham  
 Belgium  
 T: + 32-11 24 25 60  
 F: +32-11 24 25 64  
 info@medibelgium.be  
 www.medibelgium.be

medi Canada Inc / médi Canada Inc  
 104-1375 Lionel-Boulet,  
 Varennes, Québec,  
 QC Canada J3X 1P7  
 T +1 450-583-3317 / +1 800 361 3153  
 F +1 888-583-6827  
 service@medicanada.ca  
 www.medicanada.ca

MAXIS a.s.,  
 medi group company  
 Slezská 2127/13  
 120 00 Prague 2  
 Czech Republic  
 T: +420 571 633 510  
 F: +420 571 616 271  
 info@maxis-medica.com  
 www.maxis-medica.com

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medi Norway AS  
 Vestheimvegen 35  
 4250 Kopervik  
 Norway  
 T: +47 52 84 45 00  
 F: +47 52 84 45 09  
 post@medinorway.no  
 www.medinorway.no

medi Polska Sp. z o.o.  
 ul. Łąbędzka 22  
 44-121 Gliwice  
 Poland  
 T: +48-32 230 60 21  
 F: +48-32 202 87 56  
 info@medi-polska.pl  
 www.medi-polska.pl

medi Bayreuth Unipessoal Lda.  
 Rua do Centro Cultural, no. 43  
 1700-106 Lisbon  
 Portugal  
 T +351 21 843 71 60  
 F +351 21 847 08 33  
 medi.portugal@medibayreuth.pt  
 www.medi.pt

UK Responsible Person  
 medi UK Ltd.  
 Plough Lane  
 Hereford HR4 0EL  
 Great Britain  
 T +44 1432 37 35 00  
 F +44 1432 37 35 10  
 enquiries@mediuk.co.uk  
 www.mediuk.co.uk

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### Important notes

This medical device **MD** is made only for multiple uses on a single patient. If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If you experience excessive pain or any discomfort whilst wearing, remove the product and immediately contact your doctor or medical supply store. Only wear the product on unbroken skin or over injured skin that has been medically treated, not directly on injured or damaged skin, and only after previous consultation with a medical professional.

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medi Trading (Shanghai) Co. Ltd.  
Room 613, HnR Group Building  
No.1178, Tian Yao Qiao Road  
Xuhui District, Shanghai, 200232  
People's Republic of China  
T +86-21 50582318  
infocn@medi.cn  
www.medi.cn

medi Sweden AB  
Box 6034  
192 06 Sollentuna  
Sweden  
T +46 8 96 97 98  
F +46 8 626 68 70  
info@medi.se  
www.medi.se

Medi Turk Ortopedi Medikal Iç ve Dis Tic. AS  
Mustafa Kemal Mah. 2156 Sokak No: 16 /4-5  
06510 Çankaya Ankara  
Turkey  
T: +90 312 435 20 26  
F: +90 312 434 17 67  
info@medi-turk.com  
www.medi-turk.com

medi Ukraine LLC  
Evhena Sverstiuka str, 11.  
Kiev 02002  
Ukraine  
T: +380 44 591 11 63  
F: +380 44 392 73 73  
info@medi.ua  
http://medi.ua

medi USA L.P.  
6481 Franz Warner Parkway  
Whitsett, N.C. 27377-3000  
USA  
T +1-336 4 49 44 40  
F +1-888 5 70 45 54  
info@mediusa.com  
www.mediusa.com

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# protect.Epi

## Intended purpose

protect.Epi is an elbow compression  bandage with pads.

## Indications

All indications in which soft tissue compression with profile pads at the elbow is necessary, such as:

- Lateral epicondylitis or medial epicondylitis
- Osteoarthritis and arthritis

## Contraindications

None known at present.

## Risks / Side Effects

If the aids are too tight, this can cause local pressure sores or constriction of blood vessels or nerves. Therefore you should consult the doctor treating you before you use it if the following circumstances apply to you:

- Condition or injury of the skin in the area of application, particularly with infectious indications (excessive heat, swelling or reddening)
- Paraesthesia and circulatory disorders (e.g. as seen in diabetes, varicose veins)
- Lymph drainage disorders – equally, swelling of soft tissue adjacent to the area of application which is not related to a condition

When wearing snug-fitting aids, this can result in localised skin inflammation or irritation that is due to mechanical irritation of the skin (particularly in conjunction with sweating) or due to the composition of the material.

## Intended users and patient target groups

Healthcare professionals and patients, including persons who play a supporting role in care, are the intended users for this device, provided that they have received briefing on this from healthcare professionals.

Patient target group: Healthcare professionals should provide care to adults and children, applying the available information on the measurements/sizes and required functions/indications, in line with the information provided by the manufacturer, and acting under their own responsibility.

## Note on wearing

Supports have the greatest effect during physical activity. In principle, the support can be worn throughout the day. During long periods of rest (e.g. sleeping), the support should be removed.

## Instructions for fitting

- Grasp the upper end of the support and pull this over the elbow till the crook of the arm is in the middle of the support (Fig. 1+2).
- The support is sitting properly when the internal and external bony prominences (epicondyles) are readily palpable through the gaps in the pads (Fig. 3).
- The strap enables the pressure to be regulated for each individual, depending on what their body requires (tightening, loosening). If required, the strap can also easily be removed whilst the device is being worn (Fig. 4).

---

### Care instructions

Soap residues, lotions and ointments can cause skin irritation and material wear.

- Wash the product by hand, preferably using a medi clean detergent, or in delicate cycle at 30°C using a mild detergent without fabric conditioners.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

---

### Storage instructions

Keep the product in a dry place and do not expose to direct sunlight.

---

### Material composition

Polyamide, Elastane

### Liability

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

### Disposal

Dispose of properly after use.

In the event of any complaints regarding the product such as damage to the fabric or a fault in the fit, please report to your specialist medical retailer directly. Only serious incidents which could lead to a significant deterioration in health or to death are to be

reported to the manufacturer or the relevant authorities in the EU member state. The criteria for serious incidents are defined in Section 2, No. 65 of the Regulation (EU) 2017/745 (MDR).

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