medi GmbH & Co. KG
Medicusstraße 1
95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-780
ortho@medi.de
www.medi.de

medi Australia Pty Ltd
Unit 4/13-21 Hallmark Street
Pendle Hill NSW 2145
Australia
T +61 2 9890 8696
F +61 2 9890 8439
sales@medi.au
www.medi.australia.com.au

medi Austria GmbH
Adamgasse 16/7
6020 Innsbruck
Austria
T +43 512 57 95 15
F +43 512 57 95 15 45
vertrieb@medi-austria.at
www.medi-austria.at

medi Belgium NV
Staatsbaan 77/0099
3945 Ham
Belgium
T: +32-11 24 25 60
F: +32-11 24 25 64
info@medibelgium.be
www.medibelgium.be

medi Canada Inc / médi Canada Inc
104-1375 Lionel-Boulet,
Varenes, Québec,
QC Canada J3X 1P7
T +1 450-583-3317 / +1 800 361 3153
F +1 888-583-6827
service@medicanada.ca
www.medicanada.ca

MAXIS a.s.,
medi group company
Slezská 2127/13
120 00 Prague 2
Czech Republic
T: +420 571 633 510
F: +420 571 616 271
info@maxis-medica.com
www.maxis-medica.com

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medi Trading (Shanghai) Co. Ltd.
Room 613, HnR Group Building
No.1178, Tian Yao Qiao Road
Xuhui District, Shanghai, 200232
People's Republic of China
T +86-21 50582318
info@medi.cn
www.medi.cn

UK Responsible Person
medi UK Ltd.
Plough Lane
Hereford HR4 0EL
Great Britain
T +44-1432 37 35 00
F +44-1432 37 35 10
enquiries@mediuk.co.uk
www.mediuk.co.uk

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Important notes

This medical device **MD** is made only for multiple uses on a single patient. If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If you experience excessive pain or any discomfort whilst wearing, remove the product and immediately contact your doctor or medical supply store. Only wear the product on unbroken skin or over injured skin that has been medically treated, not directly on injured or damaged skin, and only after previous consultation with a medical professional.

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Polyamid, Elasthan, Polyester, Viskose

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Lumbamed® basic

Intended purpose

Lumbamed basic is a brace for stabilisation  of the lumbar spine.

Indications

All indications requiring stabilisation of the lumbar spine, such as:

- Lower back pain (lumbago)
- Muscular insufficiency
- Painful conditions of the lumbosacral junction

Contraindications

None known at present.

Risks / Side Effects

If the aids are too tight, this can cause local pressure sores or constriction of blood vessels or nerves. Therefore you should consult the doctor treating you before you use it if the following circumstances apply to you:

- Condition or injury of the skin in the area of application, particularly with infectious indications (excessive heat, swelling or reddening)
- Paraesthesia and circulatory disorders (e.g. as seen in diabetes, varicose veins)
- Lymph drainage disorders – equally, swelling of soft tissue adjacent to the area of application which is not related to a condition

When wearing snug-fitting aids, this can result in localised skin inflammation or irritation that is due to mechanical irritation of the skin (particularly in conjunction with sweating) or due to the composition of the material.

Intended users and patient target groups

Healthcare professionals and patients, including persons who play a supporting role in care, are the intended users for this device, provided that they have received briefing on this from healthcare professionals.

Patient target group: Healthcare professionals should provide care to adults and children, applying the available information on the measurements/sizes and required functions/indications, in line with the information provided by the manufacturer, and acting under their own responsibility.

Note on wearing

Supports have the greatest effect during physical exercise. In principal, the support can be worn all day. The special cut in the area of the groin is decisive for the correct and comfortable fit of the support. However, this depends on how comfortable the support is to wear. The support should be taken off for longer breaks (e.g. sitting for a long time or a long car drive, sleeping).

The support is also suitable for use in water. After use, please rinse out the product thoroughly.

Fitting instructions

Use the “magenta-coloured edge” (bottom border) as a guide when putting on the brace.

- First, slide your left hand into the inside hand loop on the left side of the fastening (fig. 1).
- Wrap the brace around your body and slide your right hand into the outer hand loop on the right side of the fastening (fig. 2+3).
- Pull both fastening sides together in

the middle so that that they overlap as evenly as possible (fig. 4).

- The brace is positioned correctly when the bottom fastening edge is just above the pubic bone (fig. 5).
- The small finger tab on the edge of the right side of the fastener will help you to undo the brace again (fig. 6).

Care instructions

Hook and loop fasteners should be closed for washing. We recommend using a laundry bag. Fabric softeners, fats, oils, lotions, ointments and soap residues can affect the material, hinder the Clima Comfort effect and cause skin irritations.

- You can wash the product by hand, preferably using a medi clean detergent, or in delicate cycle at 30°C using a mild detergent without fabric conditioners.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

Storage instructions

Keep the product in a dry place and do not expose to direct sunlight.

Material composition

Polyamide, elastane, polyester, viscose

Liability

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

Disposal

Dispose of properly after use.

Your medi Team

Wishes you a rapid recovery!

In the event of any complaints regarding the product such as damage to the fabric or a fault in the fit, please report to your specialist medical retailer directly. Only serious incidents which could lead to a significant deterioration in health or to death are to be reported to the manufacturer or the relevant authorities in the EU member state. The criteria for serious incidents are defined in Section 2, No. 65 of the Regulation (EU) 2017/745 (MDR).

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