medi GmbH & Co. KG

Medicusstraße 1
95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-780
ortho@medi.de
www.medi.de

medi Australia Pty Ltd
83 Fennell Street
North Parramatta NSW 2151
Australia
T +61-2 9890 8696
F +61-2 9890 8439
sales@mediaustralia.com.au
www.mediaustralia.com.au

medi Austria GmbH
Adamgasse 16/7
6020 Innsbruck
Austria
T +43 512 57 95 15
F +43 512 57 95 15 45
vertrieb@medi-austria.at
www.medi-austria.at

medi Belgium NV
Staatsbaan 77/0099
3945 Ham
Belgium
T: + 32-11 24 25 60
F: +32-11 24 25 64
info@medibelgium.be
www.medibelgium.be

medi Canada Inc / médi Canada Inc
104-1375 Lionel-Boulet,
Varennes, Québec,
QC Canada J3X 1P7
T +1 450-583-3317 / +1 800 361 3153
F +1 888-583-6827
service@medicanada.ca
www.medicanada.ca

MAXIS a.s.,
medi group company
Slezská 2127/13
120 00 Prague 2
Czech Republic
T: +420 571 633 510
F: +420 571 616 271
info@maxis-medica.com
www.maxis-medica.com

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Important notes

This medical device **MD** is made only for multiple uses on a single patient . If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If you experience excessive pain or any discomfort whilst wearing, remove the product and immediately contact your doctor or medical supply store. Only wear the product on unbroken skin or over injured skin that has been medically treated, not directly on injured or damaged skin, and only after previous consultation with a medical professional.

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Spinomed® active Damen

Intended purpose

Spinomed active is a brace designed to actively relieve load and correct the lumbar spine/thoracic spine in the sagittal plane.

Indications

All indications that require stress relief on and/or correction of the lumbar/thoracic vertebrae and sagittal limitation of the range of motion such as:

- Osteoporotic fracture of the vertebral bodies of the cervical and/or lumbar spine
- Juvenile Scheuermann's disease
- Kyphosis with chronic back pain

Contraindications

None known at present.

Risks / Side Effects

If the aids are too tight, this can cause local pressure sores or constriction of blood vessels or nerves. Therefore you should consult the doctor treating you before you use it if the following circumstances apply to you:

- Condition or injury of the skin in the area of application, particularly with infectious indications (excessive heat, swelling or reddening)
- Paraesthesia and circulatory disorders (e.g. as seen in diabetes, varicose veins)
- Lymph drainage disorders – equally, swelling of soft tissue adjacent to the area of application which is not related to a condition

When wearing snug-fitting aids, this can result in localised skin inflammation or irritation that is due to mechanical irritation of the skin (particularly in conjunction with sweating) or due to the

composition of the material.

Intended users and patient target groups

Healthcare professionals and patients, including persons who play a supporting role in care, are the intended users for this device, provided that they have received briefing on this from healthcare professionals.

Patient target group: Healthcare professionals should provide care to adults and children, applying the available information on the measurements/sizes and required functions/indications, in line with the information provided by the manufacturer, and acting under their own responsibility.

How does the orthosis work?

As a consequence of the tension in the material and the support from the back splint, the trunk is elevated. The trunk musculature is activated and humpback is reduced. The muscles are unconsciously strengthened, which increases mobility and reduces pain.

Note on wearing

Your doctor will specify how often you have to wear the orthosis. While you are getting used to the orthosis (in the first two weeks), you should wear it for longer and longer periods. Depending on your state of health, it may be enough to wear it for only 30 min at a time. The period of wear can then be gradually increased. As your muscles get stronger, you may feel mild or moderate cramp. Once your strength has increased, the cramp slowly decreases, till it has totally disappeared. The cramp is a sign that the orthosis is working! You must take the orthosis off at night. During extended resting periods or when you are sitting, you can simply remove the splint from the back pocket, without having to remove the body.

Please contact your doctor if you suffer from continuous muscle pain, in spite of using the orthosis properly.

Fitting instructions

- Your orthopaedics dealer has premoulded the splint. Push the splint into the back pocket. The ribs must point outwards. The narrower end is then on top. If you find it easier, the splint can be pushed in at the end.
- Open the zip with the hooks in front.
- Get into the body. First close the hooks and then the zip.
- If necessary, the length of the body can be adjusted with the hooks on the carrier and the fastener in the gusset.
- It may be necessary to check the body and splint from time to time. Please get your orthopaedics dealer to do this for you.

Care instructions

The back support has to be removed before washing. Please close any fasteners before washing (hook and loop, zipper, hooks and eyes). Soap residues, lotions and ointments can cause skin irritation and material wear.

- Wash the product by hand, preferably using a medi clean detergent, or in delicate cycle at 30°C using a mild detergent without fabric conditioners.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

To avoid discoloration, always wash the body with similar colours.

Storage instructions

Keep the product in a dry place and do not expose to direct sunlight.

Material composition

Aluminium, Polyamide, Elastane, cotton

Liability

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

Disposal

The product can be disposed of in the domestic waste.

Your medi team

wishes you a successful therapy!

In the event of any complaints regarding the product such as damage to the fabric or a fault in the fit, please report to your specialist medical retailer directly. Only serious incidents which could lead to a significant deterioration in health or to death are to be reported to the manufacturer or the relevant authorities in the EU member state. The criteria for serious incidents are defined in Section 2, No. 65 of the Regulation (EU) 2017/745 (MDR).

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