medi GmbH & Co. KG
Medicusstraße 1
95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-780
ortho@medi.de
www.medi.de

medi Australia Pty Ltd
83 Fennell Street
North Parramatta NSW 2151
Australia
T +61-2 9890 8696
F +61-2 9890 8439
sales@mediaustralia.com.au
www.mediaustralia.com.au

medi Austria GmbH
Adamgasse 16/7
6020 Innsbruck
Austria
T +43 512 57 95 15
F +43 512 57 95 15 45
vertrieb@medi-austria.at
www.medi-austria.at

medi Belgium NV
Staatsbaan 77/0099
3945 Ham
Belgium
T: + 32-11 24 25 60
F: +32-11 24 25 64
info@medibelgium.be
www.medibelgium.be

medi Canada Inc / médi Canada Inc
104-1375 Lionel-Boulet,
Varennnes, Québec,
QC Canada J3X 1P7
T +1 450-583-3317 / +1 800 361 3153
F +1 888-583-6827
service@medicanada.ca
www.medicanada.ca

MAXIS a.s.,
medi group company
Slezská 2127/13
120 00 Prague 2
Czech Republic
T: +420 571 633 510
F: +420 571 616 271
info@maxis-medica.com
www.maxis-medica.com

medi Danmark ApS
Vejlegardsvej 59
2665 Vallensbaek Strand
Denmark
T +45-70 25 56 10
kundeservice@sw.dk
www.medidanmark.dk

medi Bayreuth Espana SL
C/Canigo 2-6 bajos
Hospitalet de Llobregat
08901 Barcelona
Spain
T +34-932 60 04 00
F +34-932 60 23 14
medi@mediespana.com
www.mediespana.com

medi France
Z.I. Charles de Gaulle
25, rue Henri Farman
93297 Tremblay en France Cedex
France
T +33-1 48 61 76 10
F +33-1 49 63 33 05
infos@medi-france.com
www.medi-france.com

medi Hungary Kft.
Bokor u. 21.
1037 Budapest
Hungary
T +36 1 371-0090
F +36 1 371-0091
info@medi.hu
www.medi.hu

medi Japan K.K.
5-1 Nihonbashi Kabutocho Chuoku
103-0026 Tokyo
Japan
T: +81 3 6778 2590
F: +81 3 5847 7901
info@medi-japan.jp
www.medi-japan.co.jp

medi Nederland BV
Heusing 5
4817 ZB Breda
The Netherlands
T +31-76 57 22 555
F +31-76 57 22 565
info@medi.nl
www.medi.nl

medi Polska Sp. z o.o.
ul.tabędzka 22
44-121 Gliwice
Poland
T: +48-32 230 60 21
F: +48-32 202 87 56
info@medi-polska.pl
www.medi-polska.pl

medi Bayreuth Unipessoal, Lda
Rua do Centro Cultural, no. 43
1700-106 Lisbon
Portugal
T +351-21 843 71 60
F +351-21 847 08 33
medi.portugal@medibayreuth.pt
www.medi.pt

UK Responsible Person
medi UK Ltd.
Plough Lane
Hereford HR4 0EL
Great Britain
T +44-1432 37 35 00
F +44-1432 37 35 10
enquiries@mediuk.co.uk
www.mediuk.co.uk

UK
CA

E012659 / 07.2024

Lumbamed® sacro

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Important notes

This medical device **[MD]** is made only for multiple uses on a single patient. If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If you experience excessive pain or any discomfort whilst wearing, remove the product and immediately contact your doctor or medical supply store. Only wear the product on unbroken skin or over injured skin that has been medically treated, not directly on injured or damaged skin, and only after previous consultation with a medical professional.

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medi Trading (Shanghai) Co. Ltd.
Room 1703, T1, CES WEST BUND CENTER,
No. 277, Long Lan Road
Xuhui District
200232 Shanghai
People's Republic of China
T: +86-21 50582319
F: +86-21 50582319

Medi Turk Ortopedi Medikal İc ve Dis Tic. AS
Mustafa Kemal Mah. 2156 Sokak No: 16 /4-5
06510 Çankaya Ankara
Turkey
T: +90 312 435 20 26
F: +90 312 434 17 67
info@medi-turk.com
www.medi-turk.com

medi Ukraine LLC
Evhena Sverstiuka str, 11.
Kiev 02002
Ukraine
T: +380 44 591 11 63
F: +380 44 392 73 73
info@medi.ua
http://medi.ua

medi USA L.P.
6481 Franz Warner Parkway
Whitsett, N.C. 27377-3000
USA
T +1-336 4 49 44 40
F +1-888 5 70 45 54
info@mediusa.com
www.mediusa.com

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Bandage: Polyamid, Polyester, Elasthan, Viskose

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Lumbamed® sacro

Intended purpose

Lumbamed sacro is a brace for stabilising  the pelvis.

Indications

All indications which require stabilisation of the pelvis, such as:

- Sacroiliac joint dysfunction
- Sacroiliac joint instability
- Muscle pain and tendon disease in the pelvic region
- Pelvic ring instability
- For recurring muscle and tendon disease (rectus abdominis muscle, piriformis muscle)

Contraindications

None known at present.

Risks / Side Effects

If the aids are too tight, this can cause local pressure sores or constriction of blood vessels or nerves. Therefore you should consult the doctor treating you before you use it if the following circumstances apply to you:

- Condition or injury of the skin in the area of application, particularly with infectious indications (excessive heat, swelling or reddening)
- Paraesthesia and circulatory disorders (e.g. as seen in diabetes, varicose veins)
- Lymph drainage disorders – equally, swelling of soft tissue adjacent to the area of application which is not related to a condition

When wearing snug-fitting aids, this can result in localised skin inflammation or irritation that is due to mechanical irritation of the skin (particularly in conjunction with sweating) or due to the composition of the material.

Intended users and patient target groups

Healthcare professionals and patients, including persons who play a supporting role in care, are the intended users for this device, provided that they have received briefing on this from healthcare professionals.

Patient target group: Healthcare professionals should provide care to adults and children, applying the available information on the measurements/sizes and required functions/indications, in line with the information provided by the manufacturer, and acting under their own responsibility.

Note on wearing

Supports are particularly beneficial during physical exercise. The supports is designed to fit comfortably and may be worn all day. It should be removed during prolonged periods of inactivity (e.g. sitting for a long time or a long car drive, sleeping).

The support is suitable for use in water. After use, please rinse thoroughly.

Putting on / taking off

- Before you put the brace on, loosen the fastening straps so that the brace can be put on easily. Release the belts' fasteners and fix them loosely at the back of the belt, on the velour.
- Fix the pads in the middle of and/or according to the doctor's or specialist's instructions on the grey hook and loop. If no other advice has been given, the pads should be fixed with the narrow end pointing downwards (Fig. 1 & 2).
- The label on the right inside section provides instructions on how to put the brace on (Fig. 3).
- First, slip your hand into the hand strap on the left interior fastening side.
 - Loop the brace around your pelvis

- (Fig. 4).
- Now, slip your hand into the outer hand strap on the right-hand side (Fig. 5).
 - The brace should be fastened with the bottom edge of the fastener sitting just above the pubis. Pull the two fastening sides over each other and fasten them as centrally as possible (Fig. 6).
 - Then release the fastening straps' fasteners and pull these to the front simultaneously. You can adjust the tightness of the brace according to your needs and level of pain (Fig. 7 + 8).
 - To check the position of the brace, lift your thigh up slightly. The bottom of the fastening strap should be level with your groin (Fig. 9).
 - You can remove parts of the brace and just wear the outer, non-elastic component with the fastening strap system (Fig. 12). For this, please loosen the hook and loop fastenings on both sides of the fastening interior side and remove the elastic knitted fabric (Fig. 10).
 - You can also place the pads in the centre of the belt here or position them here according to your doctor's or a specialist's instructions, if required (Fig. 11).
 - Then put the brace on as described above (Fig. 12).

Our recommendation: Wear the brace directly over underwear, or a very thin layer of clothing, so that the massage pads can have an optimal effect.

If you also wish to wear this brace at night, please observe the additional instructions for use for the Fastening Extension.

There is no data on the use of Lumbamed sacro by pregnant

women. Therefore, only doctors may decide whether this brace is appropriate, and doctors must be consulted for clarification in individual cases.

Instructions for care

The pads should be removed before washing. If necessary, the pads can be cleaned with a dampened cloth. Please fasten all hook and loop fastenings before washing. We recommend that you use a laundry bag. Fabric softeners, fats, oils, lotions, ointments and soap residues can affect the material, hinder the Clima Comfort effect and cause skin irritations.

- Wash the product by hand, preferably with mediclean soap, or in a delicate wash cycle at 30°C with mild detergent and without fabric conditioner.
- Allow to air-dry.
- Do not bleach.
- Do not iron.
- Do not dry-clean.

Storage instructions

Keep the product in a dry place and do not expose to direct sunlight.

Material composition

Band: Polyamide, polyester, elastane, viscose

Pads: Polyurethane

Liability

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in these instructions for use.

Disposal

You can dispose of this product with your household waste.

**Your medi Team**

wishes you a speedy recovery!

In the event of any complaints regarding the product such as damage to the fabric or a fault in the fit, please report to your specialist medical retailer directly. Only serious incidents which could lead to a significant deterioration in health or to death are to be reported to the manufacturer or the relevant authorities in the EU member state. The criteria for serious incidents are defined in Section 2, No. 65 of the Regulation (EU) 2017/745 (MDR).

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